

---

## >> Test Type

Accepted for the following test types:

- Carrier Screening
- Hereditary Cancer Screening

---

## >> Specimen Requirements

### Specimen type

- Saliva
- We are unable to accept:
  - » Saliva in any container other than the Oragene™ kits
  - » Buccal swabs and mouthwash specimens
  - » Saliva from patients who have had a bone marrow transplant or stem cell transplant
  - » Saliva from patients who had a blood transfusion less than two weeks prior to specimen collection
  - » Specimen received after stability date
  - » Specimen received frozen

### Volume

Minimum volume 1.5-2 mL

### Container

Oragene™ self collection kit (OGD610)

### Transport Temperature

- Room temperature
- Do not freeze

### Specimen Stability

120 Hours

---

## >> Labeling

Each tube must be labeled with:

- The patient's full name
- Date of birth

The patient name and date of birth on each tube must match what is on the test requisition.

---

---

## >> Documentation

Samples cannot be processed without the following:

- A completed and signed copy of the test requisition form
- A signed patient consent form
- Copy of the front and back of the insurance card

---

## >> Shipping Requirements

Depending on your location, specimens should be shipped via FedEx Priority Overnight or through the local courier service.

**To assist with specimen shipping, including scheduling a pick-up, please call Client Services at 888-530-2239**

Saliva should be shipped at room temperature.

### Shipping address

Attn: Client Services  
50 Tice Blvd, Suite 183  
Woodcliff Lake, NJ 07677

If you are sending more than one sample per day you can include multiple kits in the same FedEx package.

PsiGenex accepts packages Monday through Saturday.

---